

## 1. Introduction

You have just received the Table alarm with mat by Vahlkamp. The Table alarm has been especially designed for individuals with hand function difficulties. The user need only press the alarm unit slightly in order to activate the (existing) nurse call system.

### 1.1 Components

The Table alarm consists of the following components (see illustration 1) :

1. **Alarm unit**, mat, size 170x170x4mm
2. **Transmitter unit**, connected with a cable to the alarm mat
3. **Receiver unit**, with 220 Volt adapter and cable connection for the (existing) call system

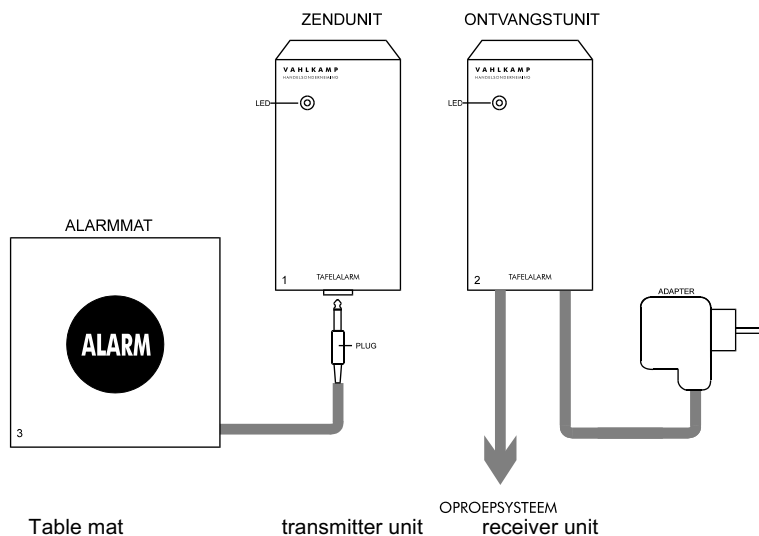


Illustration 1: components Table alarm with mat

## 2. Installation

### 2.1 Receiver unit

You are to connect the grey cable to a plug that is suitable for your nurse alarm system.

The following connections apply to the grey cable:

- brown: NO
- green: NC
- white: COM

*Example:*

If your nurse alarm system has a make-contact, then connect white and brown.  
You are to connect white and green in the case of a breaker contact.

Plug the adapter into a 220-240 Volt wall socket and connect the plug to the (existing) call system.

### 2.2 Transmitter unit

Attach the transmitter unit at the bottom of the table.

### 2.3 Alarm mat

Put the mat on the table and connect the mat with the transmitter unit.

The installation is now complete. The system can be tested by applying slight pressure to the centre of the mat.

### 2.4 Testing

Install the receiver unit as described above and press on the Alarm mat.

The Led of the transmitter unit will light up for red for 5 seconds as an indication that the emergency call has been send.

The Led of the receiver unit will light up red and buzz, to indicate that the emergency call has been received and the (existing) nurse call system will be activated. After 12 seconds, the Led will light up green and the buzzer stops buzzing to indicate that the receiver is ready to receive a new emergency call.

The Table alarm is ready for use once the steps above have been completed.

## **GUARANTEE**

Vahlkamp observes a term of guarantee of 12 months if the product is used normally. Repairs are free of charge during this period.

The guarantee does not apply to defects that are the result of improper use, breakage, a fall, dismantling, fire and/or water damage, neglect or incompetent repairs. Batteries are not covered by the guarantee.

Should the occasion arise, then the buyer is to contact Vahlkamp, which will then see to the repairs.

