1. Introduction

You have just received the Wireless Floor alarm by Vahlkamp. We will first point out the advantages of this system before providing directions for the installation and use.

PLUSPUNTEN

The wireless Floor alarm is a user friendly product and it offers the following advantages:

- * **Extensive testing possibilities / battery check.**
 - The switch is turned ON: the LED lights up in red for an instant
 - The contact with the mat is made: the LED will light up for 5 seconds.

- * **No inconvenient cords or wires.**

The Chair alarm is wireless. And so the chair can be positioned at a distance of 5 to 10 metres from the nurse alarm system. You will not be hindered by cords or wires on the floor.

1.1 Components

The wireless Floor alarm consists of the following components (see illustration 1):

1. **Transmitter unit** with on/off switch
2. **Receiver unit**, with 220-240 Volt adapter and cable for connection with the nurse call system
3. **Floor mat**, grey mat 55 x 55 cm with accompanying cable and plug and with non-slip underneath.

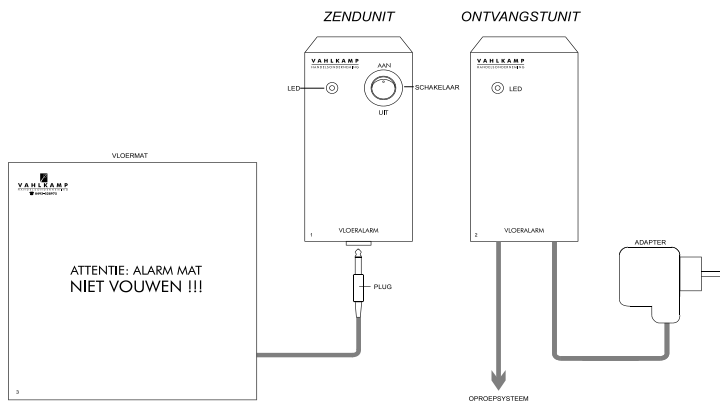
Illustration 1: Components wireless Floor alarm

2. Installation

2.1 Receiver unit

You are to connect the grey cable to a plug that is suitable for your nurse alarm system.

The following connections apply to the grey cable:

- brown: NO
- green: NC
- white: COM

Example:

If your nurse alarm system has a make-contact, then connect white and brown.

You are to connect white and green in the case of a breaker contact.

The transmitter unit must then be permanently mounted on the wall near the alarm system connection.

Plug the adapter into a 220-240 Volt wall socket and connect the plug to the nurse alarm system. The receiver unit is now able to relay the alarm signal to the alarm system.

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